




Action Assist

Specialist bed supplier

www.actionassist.co.uk

Countess Handset Instructions

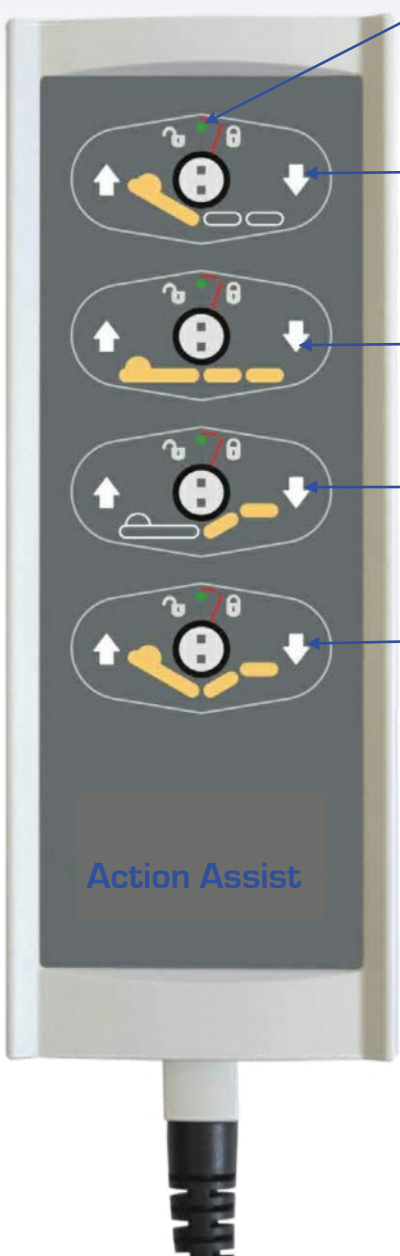

Power LED.
Green - Unlocked.
Yellow - Locked.

Back rest
Buttons.

High - Low
Buttons.

Knee break
Buttons.

Simultaneous control
Of the Back rest
& knee break.

Locking Key



Key to lock/
unlock each
function
individually

**PLEASE READ THIS
OPERATING GUIDE
CAREFULLY AND
ALWAYS FOLLOW
THE INSTRUCTIONS
ON THE REVERSE
OF THIS GUIDE.**

AFSSAPS

Recommendations 24/07/02


If you have any questions regarding the functioning of the handset, or in case of fault, please call the phone number below.


Action Assist Ltd
Unit 6 Moor Lane Trading Estate
Sherburn In Elmet
Leeds
North Yorkshire
LS25 6ES
Tel: 01977 689400
Fax: 01977 689401

Email: info@actionassist.co.uk
Web: www.actionassist.com

Action Assist

Handset with mechanical locking key

- *Shown on the front of the handset are 4 locking possibilities one per function.*
- *By default the handset function are unlocked and the power Indicator is Green.*
- *To lock a function, insert the key into the relevant function and turn it towards the locked position indicated by the symbol *
- *To unlock, insert the key into the relevant function and turn it towards the unlocked position (open symbol) and stop turning in the middle position. NB: Do not force. The power indicator will turn green.*

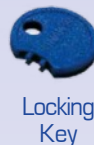

NB: The mechanical key is a safety advise, to be used by the carer, its purpose is to lock the electrical functions when necessary. If locking of the functions is not required leave the key attached to the cable. In the event of the key being damaged or misplaced a replacement key can be purchased from our service department.

Safety Instructions

- ➔ The bed should always be plugged into a power supply without any adapters. If it is necessary to use an extension lead, it must comply with the ratings for the bed (220 - 230 volts, 50/60 hertz).
- ➔ Before handling the bed, its mains plug must always be disconnected from the power supply. When the bed is to be moved ensure the power lead is wrapped around the cable tidy to prevent it making contact with the floor or castors.
- ➔ When moving the bed, ensure the actuator and handset leads do not become trapped and prevent the leads becoming tangled.
- ➔ When the bed is being used, ensure at least two of the four locking castors are locked.
- ➔ Keep the bed clean at all times, if the bed becomes soiled it can be cleaned with a wet cloth or sponge and soap or a disinfectant wipe.
- ➔ Cables should be inspected on a regular basis for any damage, if damage is noticed or there are any other technical issues, contact our technicians on the number shown below.

Your bed is a medical device, intended to improve care, your well-being and your comfort. Therefore it should be maintained in a good condition and used only for what it is designed for. The bed should not be operated by young children, only by authorised users.

