


Handset with key lock function

• When the mains lead is connected to a live power outlet the power - On indicator illuminates. However if Indicator A does not illuminate, the handset functions are locked.

• To unlock the handset functions, insert the key into the lock positioned on the back of the handset and turn anti-clockwise. Indicator A will now illuminate enabling all handset functions to operate.

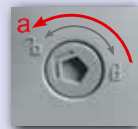

• To re-lock the handset functions insert the key into the lock and turn clockwise, indicator A will extinguish, meaning all handset functions are Locked.



NB: The key is a safety device, to be used by the carer, its purpose is to lock all electrical functions when necessary. If locking of the functions is not required leave the key attached to the cable. In the event of the key being damaged or misplaced a replacement key can be purchased from our service department.


Safety Instructions

- ➔ The bed should always be plugged into a power supply without any adapters. If it is necessary to use an extension lead, it must comply with the ratings for the bed (220 - 230 volts, 50/60 hertz).
- ➔ Before handling the bed, its mains plug must always be disconnected from the power supply. When the bed is to be moved ensure the power lead is wrapped around the cable tidy to prevent it making contact with the floor or castors.
- ➔ When moving the bed, ensure the actuator and handset leads do not become trapped and prevent the leads becoming tangled.
- ➔ When the bed is being used, ensure at least two of the four locking castors are locked.
- ➔ Keep the bed clean at all times, if the bed becomes soiled it can be cleaned with a wet cloth or sponge and soap or a disinfectant wipe.
- ➔ Cables should be inspected on a regular basis for any damage, if damage is noticed or there are any other technical issues, contact our technicians on the number shown Overleaf.

Your bed is a medical device, intended to improve care, your well-being and your comfort. Therefore it should be maintained in a good condition and used only for what it is designed for. The bed should not be operated by young children, only by authorised users.



Countess Handset Instructions

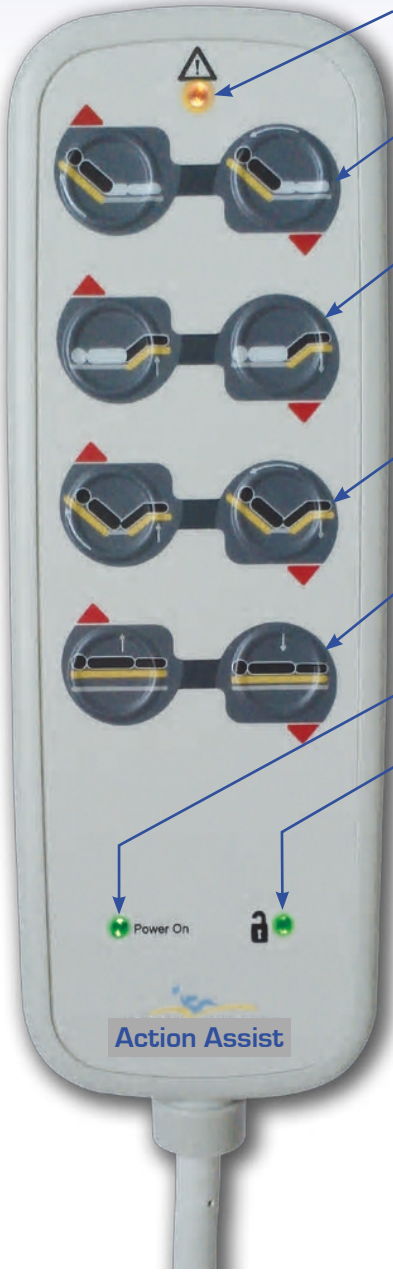

FUNCTION IN USE INDICATOR

BACK REST SECTION CONTROL

LEG SECTION CONTROL

BACK REST & LEG SECTION SIMULTANEOUS CONTROL

VARIABLE HEIGHT ADJUSTMENT CONTROL

POWER ON INDICATOR

INDICATOR A

Lockout with key



Handset with function lock key inserted

PLEASE READ THIS OPERATING GUIDE CAREFULLY AND ALWAYS FOLLOW THE INSTRUCTIONS ON THE REVERSE OF THIS GUIDE.

*AFSSAPS
Recommendations 24/07/02*

If you have any questions regarding the functioning of the handset, or in case of fault, please call the phone number below:

Action Assist Ltd
Unit 6 Moor Lane Trading Estate
Sherburn In Elmet
Leeds
North Yorkshire
LS25 6ES
Tel: 01977 689400 Fax: 01977 689401

Email: info@actionassist.co.uk
Web: www.actionassist.com