


## Reprogramming Procedure for the Princess 5000 lateral tilting bed.

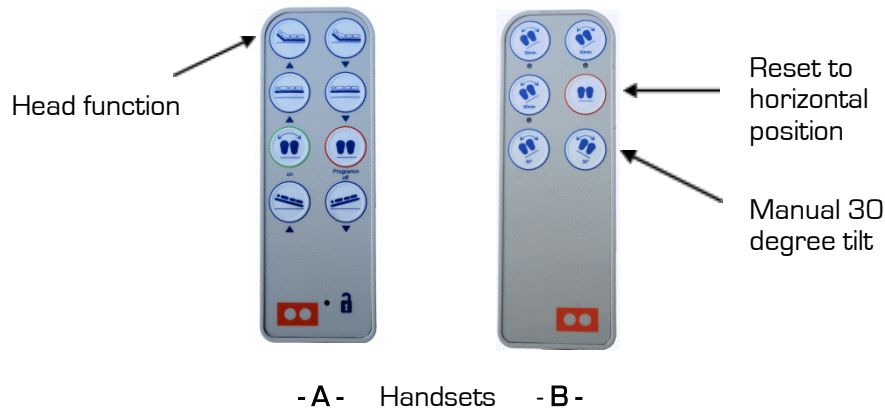

### Reprogramming Procedure:

Failure of the backrest and lateral tilt functions to operate may occur due to a break or surge in the power supply. To rectify, please use the following reprogramming procedure.

Remove the power plug from mains outlet. Leave unplugged for a minimum of 30 seconds. Hold handset **A**, press the top left hand button (Head Function: up) once.

Replace the power plug into mains outlet and switch on. Hold handset **B**, continually press the bottom right hand button (manual 30 degree tilt) until fully tilted, now press the button directly above (button with the red outline) once, to reset to the sleeping platform to the horizontal position.

The bed will now be reprogrammed and all functions should work correctly. If you are experiencing any further problems please contact our service department on the telephone number shown below..

**NB:** Some beds have a battery backup for emergency lowering of the back rest/bed. Before the reprogramming procedure is undertaken this battery should be disconnected from the control box. The battery is located on top of the main control box and is attached by a grey lead that plugs back into the control box body.

To disconnect the battery, raise the whole bed to its highest position and raise the side rail. The main control box is positioned under the back rest of the bed. The battery lead can be disconnected from underneath the bed by pulling the grey plug out of the control box body, as shown in the following photographs.



Battery connection.

Remove by pulling.

Battery connection removed.

Once the bed is reprogrammed and fully functioning the battery should be reconnected.