




Action Assist

Specialist bed supplier

www.actionassist.co.uk

Viscount 300 Chair Bed Handset Instructions

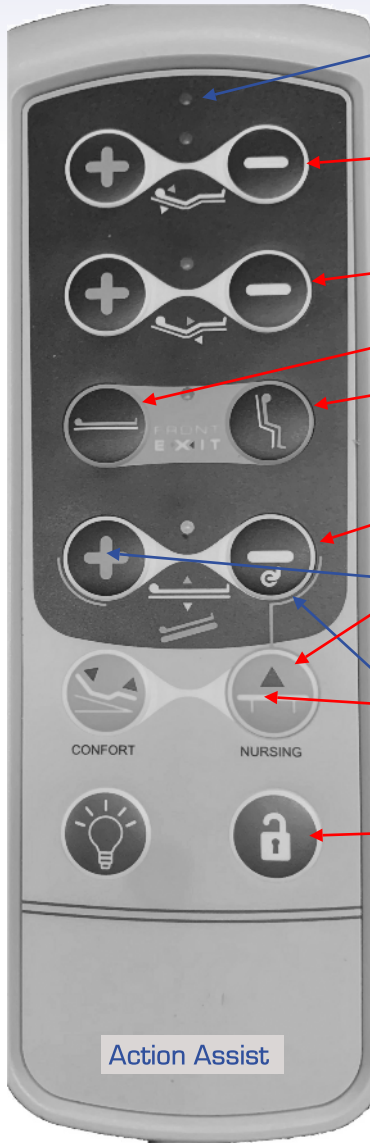

FUNCTION IN USE
INDICATOR

BACK REST SECTION
CONTROL

LEG SECTION
CONTROL

SLEEPING POSITION

CHAIR POSITION

VARIABLE HEIGHT
ADJUSTMENT CONTROL

REVERSE
TRENDELENBURG
(Press both buttons)

TRENDELENBURG
(Press both buttons)

INDIVIDUAL FUNCTION
LOCK (Press this button
simultaneously with the
function button required
to lock)



Lockout with key

Handset with
function lock
key inserted

**PLEASE READ THIS
OPERATING GUIDE
CAREFULLY AND
ALWAYS FOLLOW
THE INSTRUCTIONS
ON THE REVERSE
OF THIS GUIDE.**

If you have any questions regarding the functioning of the handset, or in case of fault, please call the phone number opposite.

Action Assist Ltd
Unit 6 Moor Lane Trading Estate
Sherburn in Elmet
Leeds
LS25 6ES

t: 01977 689400

f: 01977 689401



e: info@actionassist.co.uk

w: www.actionassist.co.uk



Height adjustment of the sleeping surface with the hand control :

The height of the frame carrying the sleeping surface can be adjusted from 33 cm to 76 cm by continually pressing the button with the symbol  and lowered by continually pressing the button with the symbol .


Adjustment of the back rest with the hand control :

The back section can be adjusted to an angle between 0° up to maximum 71° by continually pressing the button with the symbol  and lowered by continually pressing the button with the symbol .


Adjustment of the leg rest with the hand control :

The leg rest can be adjusted to an angle between 0° and 25° by continually pressing the button with the symbol  and lowered by continually pressing the button with the symbol .


Adjustment of the sleeping platform to the chair position :

With the bed in the sleeping position continually press the button with the symbol  the bed will start raising both the backrest and knee break and then start to tilt forwards until the bed is in the semi chair position. Release the button and press again, the leg section will lower to the floor creating a full chair position and stop.

Adjustment of the chair position to the sleeping position :

With the bed in the chair position continually press the button with the symbol  the sleeping platform will start to tilt backwards whilst raising the leg section. Once the tilt function stops the operator can lower the back rest and leg section to the desired sleeping position.

Trendelenburg control :

The bed can be lowered at the head end and raised at the foot end known as the Trendelenburg position by pressing the button with the symbol  this can help when pushing yourself up the bed.

Safety Instruction

- The bed should always be plugged into a power supply without any adapters. If it is necessary to use an extension lead, it must comply with the ratings for the bed (220 - 230 volts, 50/60 hertz).
- When the bed is to be moved ensure the power lead is wrapped around the cable tidy to prevent it making contact with the floor or castors.
- When moving the bed, ensure the actuator and handset leads do not become trapped and prevent the leads becoming tangled.
- When the bed is being used, ensure that the castors are locked.
- Keep the bed clean at all times, if the bed becomes soiled it can be cleaned with a wet cloth or sponge and soap or a disinfectant wipe.
- Cables should be inspected on a regular basis for any damage, if damage is noticed or there are any other technical issues, contact our technicians on the number shown below.

Your bed is a medical device, intended to improve care, your well-being and your comfort. Therefore it should be maintained in a good condition and used only for what it is designed for. The bed should not be operated by young children, only by authorised users.

Action Assist

Specialist bed supplier
www.actionassist.co.uk



Service Department

t : 01977 689400

e : support@actionassist.co.uk